



Swab Collection

Write your name and date of birth on the vial and record the date collected.

Follow instructions on swab package.

Throat: Carefully guide cotton swab all the way back into throat and swab gently. Then place swab tip to the bottom of Vial, snap off swab shaft and replace cap.

Vaginal and Other: Gently insert the cotton swab 2-3 inches and rotate. Then place swab tip to the bottom of Vial, snap off swab shaft and replace cap.

Refrigerate sample until shipment. Do not freeze.

Ship sample as soon as possible within 3 days of collection.

Shipping Instructions

US Domestic: Tuck front flap into box and seal with shipping label (included in box). Place label within dashed lines and adhere over front edge of box. Send from your most convenient shipping location, matching the carrier indicated on your return shipping label.

International: Seal box with clear tape. Ship with FedEx and select two to three day delivery. Shipping charges will apply. Deliveries can be made Monday through Friday. International shipping forms are available at www.diagnostechs.com (see Patients → Shipping Instructions).

For Express Couriers:

DiagnosTechs, Inc.
Sample Processing
840 S 333rd St
Federal Way, WA 98003 USA
425-251-0596